

Instructions for use


Product description

Impression Disinfectant Powder


Date of revision: 08/03/23 v1.6

topdental™
Bossklein®


Fast acting disinfectant for all kinds of impression material such as silicone, alginates and polyether.

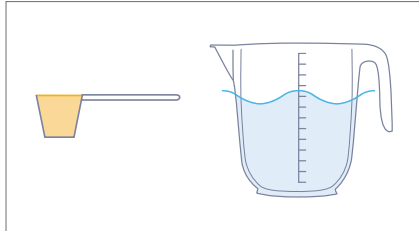


1. Dissolve 1 level measure of powder (20g ± 2g) in 980ml of warm tap water (30 - 40°C).

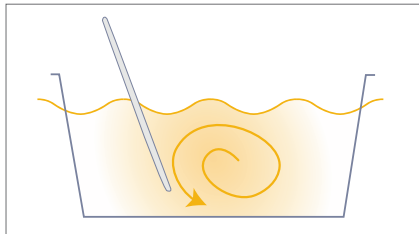


2. Mix for 60 seconds.

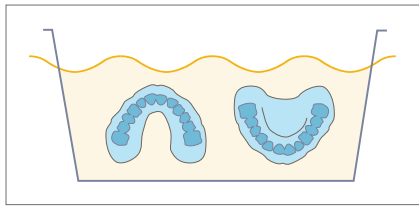

3. Place the impressions into the solution and fully immerse.

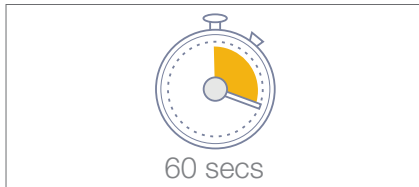

4. 60 second contact time. Do not exceed this recommended time.

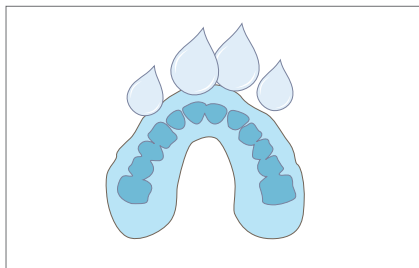

5. Rinse immediately with copious amounts of running water. Change solution daily. For professional use only. See product labels and MSDS for further information.

Efficacy: Bactericidal (60 secs), including MRSA. Yeasticidal (60 secs). Viruses to include (60 secs): Non-enveloped viruses: Murine norovirus, Adenovirus 5. Enveloped viruses: Hepatitis B virus (HBV), Hepatitis C virus (HCV), Hepatitis Delta virus, Human Immunodeficiency virus (HIV), Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles.

Tested to Standards: EN 14885, EN 14476, EN 13727, EN 13697, EN 13624.

Composition: TAED, Sodium Percarbonate, Citric Acid Monohydrate, Sodium Alkyl Aryl Sulphonate, Sodium Sulphate Anhydrous, Tetrasodium (1-hydroxyethylidene) bisphosphonate, Sodium Nitrite.

Danger. H318: Causes serious eye damage. H272: May intensify fire; oxidiser. H312: Harmful in contact with skin. P280: Wear protective gloves/protective clothing/eye protection/face protection. P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P301+P310: IF SWALLOWED: Immediately call a POISON CENTER or doctor. **Storage.** P403+P233: Store in a well-ventilated place. Keep container tightly closed. P235: Keep cool. P232: Protect from moisture. **Contraindications.** Incompatible materials: Aluminium, Copper. **Disposal.** Do not discharge into drains or rivers.

Intended Users. The products are used within the dental surgery by the dentist or dental assistant as part of their general disinfecting routine. Patients are usually not present during use, but all patients that are seen in the surgery may contribute to the need for the products.

Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.

UFI: GF10-X0N2-400J-UYDM



DIRECTA AB
P.O. Box 723, Finvids väg 8 SE-194 27
Uplands Väsby, Sweden



For additional languages
see www.topdental.co.uk



DIRECTA topdental ORSING
parkell KOHLER RÖNVIG
TrollDental Sendoline CONTACTEZ
physics For experts ceramir