

# Instructions for use

## Product description


# Daily Aspirator Cleaner and Disinfectant

Date of revision: 08/03/23 v1.6


For the daily cleaning and maintenance of dental aspirator systems.  
Can be used on wet line, dry line and semi wet/dry line suction systems.

topdental™  
Bossklein®


### 5L bottle

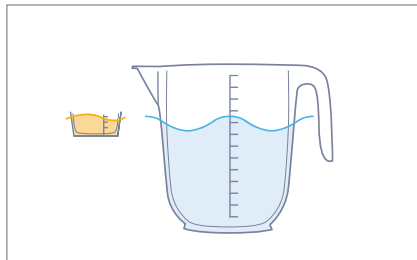

1. Add 20ml ± 1ml of concentrate to 980ml warm water (2% solution).

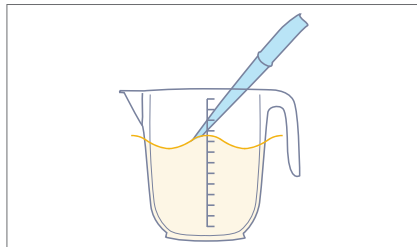


2. Aspirate via the high and low volume suction tubes.

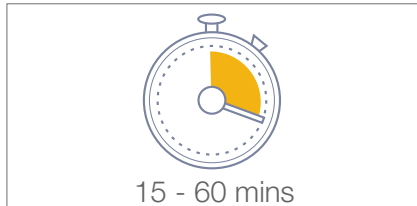


15 - 60 mins

3. Leave for a minimum of 15 minutes (to achieve full efficacy leave for 60 minutes).

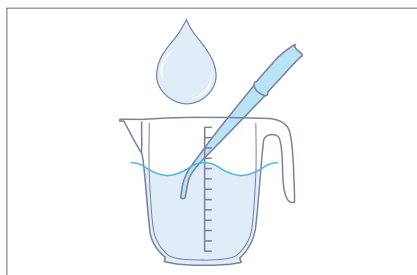


4. Rinse thoroughly with clean water.

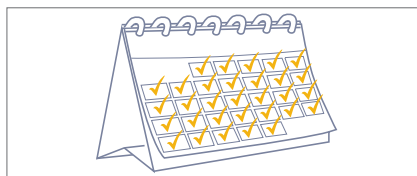


5. Use daily. Alkaline based for removal of Daily deposits. Can be used on wet line, dry line and semi wet/dry line suction systems. For professional use only. See product labels and MSDS for further information.


### 1L dispensing bottle

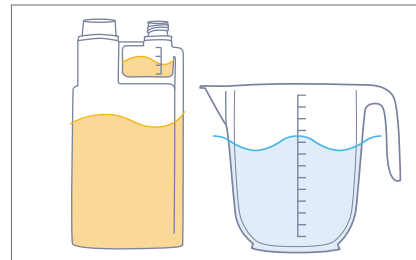

1. Add 20ml ± 1ml of concentrate to 980ml warm water (2% solution) using the dosing device on the bottle.

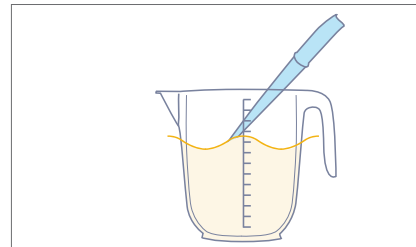


2. Aspirate via the high and low volume suction tubes.

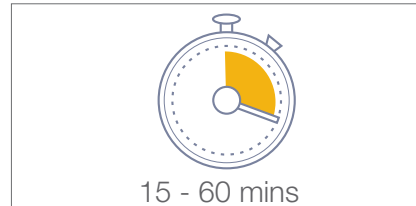


15 - 60 mins

3. Leave for a minimum of 15 minutes (to achieve full efficacy leave for 60 minutes).

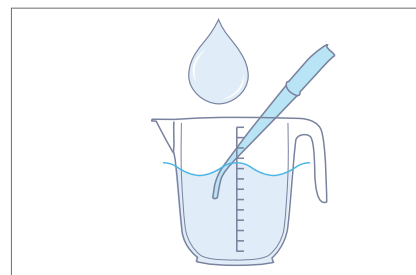

4. Rinse thoroughly with clean water.

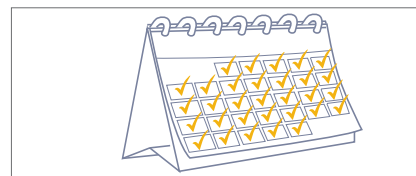

5. Use daily. Alkaline based for removal of Daily deposits. Can be used on wet line, dry line and semi wet/dry line suction systems. For professional use only. See product labels and MSDS for further information.

**Efficacy:** Bactericidal (15mins) including Salmonella typhimurium, Legionella pneumophila. Yeastcidal (15mins). Hepatitis B virus (15mins) Hepatitis C virus (15mins), Human Immunodeficiency Virus (15mins). Tuberculocidal (60mins). Fungicidal (30mins). **Tested to Standards:** EN14885, EN14476, EN14348, EN13727, EN13697, EN13624, ASTM E1052.

**Composition:** Aqua, Dimethyldioctylammonium chloride, Dipropylene glycol, 2-Aminoethanol, Tetrasodium ethylene diamine tetraacetate, Ethanol, Citric acid, Nonionic surfactant, Antifoam, Dye.

**Danger.** H314: Causes severe skin burns and eye damage. P260: Do not breathe dust/ fumes/gas/mist/vapours/spray. P280: Wear protective gloves/protective clothing/eye protection/face protection. P301+330+331: IF SWALLOWED: rinse mouth. Do NOT induce vomiting. P303+361+353: IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water. P304+340: IF INHALED: Remove person to fresh air and keep comfortable for breathing. P305+351+338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **Storage.** P403+P233: Store in a well-ventilated place. Keep container tightly closed. P235: Keep cool. **Contraindications.** Incompatible materials: Aluminium, copper. **Disposal.** Do not discharge concentrate into drains or rivers.

**Intended Users.** The products are used within the dental surgery by the dentist or dental assistant as part of their general disinfecting routine. Patients are usually not present during use, but all patients that are seen in the surgery may contribute to the need for the products.

Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.

UFI: J110-E062-M003-J8G9



For additional languages see [www.topdental.co.uk](http://www.topdental.co.uk)



Danger

DirectaDentalGroup

