



V-WIPE

Surface Disinfectant Wipe

CONTAINS ALCOHOL

ULTRA
160 PREMIUM WIPES
175 x 155mm

EN14476
VIRUS TESTED

63%
ALCOHOL
CONTENT


CONTACT
60 SEC


LEMON
AROMA

40gsm
THICKER
WIPE

160
PREMIUM
WIPES

KEY FEATURES

Bossklein V-WIPES are a powerful, alcohol based disinfectant wet wipe. Designed for the disinfection of non-invasive medical device surfaces within medical and dental environments. Ideal to use where time is critical and also in high traffic areas.

- Contact time of only 60 seconds
- Wide effective spectrum
- EN 14467 virus tested
- Kills Coronavirus
- Quick drying
- Leaves no smears or residue
- Dual registration; Medical Device & Biocide
- Fresh lemon aroma

Bossklein V-WIPES are ideal to use on alcohol resistant, non-porous surfaces within the dental surgery. The 60 second contact time makes them perfectly suited where short contact times are important, eg in between patients.

The 63% alcohol based formulation has a powerful microbial action, the perfect solution for controlling infections on resistant surfaces in clinical environments.

V-WIPES have been rigorously tested and proven to be effective against all Bacteria and Yeast. EN 14476 Virucidal testing proves efficacy against all enveloped viruses including Coronavirus, HIV, Hepatitis and Influenza in just 60 seconds.

The fresh lemon aroma leaves equipment and rooms reassuringly fresh, clean and ready for the next patient.

COMPOSITION

Ethanol (64-17-5) 63% w/w (630g/kg), Aqua, Perfumes.

INSTRUCTIONS FOR USE

Using sufficient wipes for each application wipe the clean surfaces to be disinfected so that they are completely moistened and let the disinfectant take action for 60 seconds. Allow surface moisture to evaporate. For professional use only.

REGISTRATION

Dual registered as a Class IIa medical device for use on both non invasive medical devices and as a general purpose Biocide registered disinfectant. This ensures suitability for use within all types of clinical and non clinical environments.

TEST STANDARDS

The following test standards have been achieved: EN 16615, EN 14885, EN 14476, EN 14348, EN 13727, EN 13624.

VARIANTS AND PACKAGING

BOS1000 Tub of 160 Ultra Wipes x 12 per box
BOS1001 Refill of 160 Ultra Wipes x 12 per box
BOS1002 Kit of Ultra Wipes - 6 x Refill + 1 x Tub
BOS1003 Empty Tub for Ultra Wipes x 12 per box

MICROBIOLOGICAL EFFICACY

MICROORGANISM	TIME
Bactericidal	60 sec
BS EN 13727:2012 + A2:2015	
BS EN 13697:2015 + A1:2019	
BS EN 16615:2015	
Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae, E. coli, Salmonella typhimurium, MRSA, VRE	60 sec
Yeasticidal	60 sec
BS EN 13624:2013	
Candida albicans	60 sec
Enveloped viruses	60 sec
BS EN 14476:2013 + A1 2015	
Hepatitis B virus (HBV), Hepatitis C virus (HCV), HIV, Human T Cell Leukemia virus, Measles virus, Hepatitis Delta virus (HDV), Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella virus, Rabies virus, Flavivirus, Poxviridae	60 sec
Non enveloped viruses	60 sec
BS EN 14476:2013 + A1 2015	
Adenovirus 5, Murine norovirus	60 sec
Mycobactericidal	60 sec
BS EN 14348:2005	
Mycobacterium avium	60 sec
Tuberculocidal	60 sec
BS EN 14348:2005	
Mycobacterium terrae	60 sec

Wipes are available in our "Ultra" wipe 40gsm premium substrate, for ultimate strength and grip. Each wipe measures 175x155mm and each pack contains 160 Ultra wipes. Both re usable dispenser tubs and economical refill packs are available.



Complete hygiene solutions

DirectaDentalGroup
DIRECTA topdental ORSING parkell physics
RÖNVIIG TrollDental Sendoline CONTACTEZ

Topdental (Products) Ltd
12 Ryefield Way Silsden West Yorkshire BD20 0EF UK
Tel +44 (0)1535 652 750 www.topdental.co.uk