

Instructions for use

Product description

Universal Handpiece Oil

Date of revision: 08/02/22 v1.4

topdental™
Bossklein®


All purpose Handpiece Lubricant Spray for maintenance of all dental handpieces including highspeed, slow speed and air motors. Contains white mineral oil with a trace of peppermint oil.




1. Insert into back of handpiece. Use immediately after each use and before sterilisation. Refer to handpiece maintenance instructions where applicable.



2. Spray through for one to two seconds.

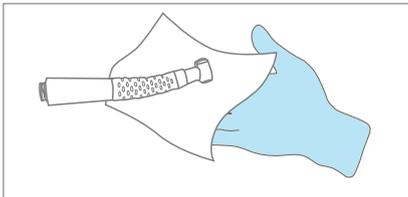

3. Wipe excess thoroughly.

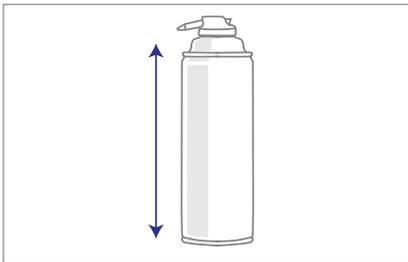

4. Hold can in a vertical position when in use. For professional use only. See product labels and MSDS for further information.

Warning. H222: Extremely flammable aerosol. H229: Pressurised container: may burst if heated. P211: Do not spray on an open flame or other ignition source.

Storage. P403+P235: Store in a well ventilated place. Keep cool. P410+P412: Protect from sunlight. Do not expose to temperatures exceeding 50°C/122°F.

Contraindications. P210: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.

Disposal. P251: Do not pierce or burn, even after use.

Intended Users. The products are used within the dental surgery by the dentist or dental assistant as part of their general routine. Patients are usually not present during use, but all patients that are seen in the surgery may contribute to the need for the products.

Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.

UFI: CW10-G031-P001-5P9X



Warning.